

TECHNICAL DATA SHEET

PHARMIA FFP2 NR

MODEL 1801.2

PERSONAL PROTECTION EQUIPMENT (PPE)

Respiratory protection mask against solid and liquid non-volatile particles, for single use. Composed of 5 layers that guarantee an effective filtration. It has elastic fixation bands, made of soft and comfortable material, and a nasal clip that is easy to shape and adjust.

It has high filtration efficiency and low resistance to breathing, being comfortable throughout the work period.

Manufacturer name:	Pharmia, Lda
Adress:	Rua Pampelido velho, 76, 4455-142 Lavra – Matosinhos
Trademark:	Pharmia FFP2 NR
Units per package:	10 unidades
Applicable legislation:	Regulamento UE 2016/425 (EPI)
Applicable standard:	EN 149:2001+A1:2019
Protection level:	FFP2 NR
Notified Body:	UNIVERSAL UYGUNLUK DEĞERLENDİRME HİZMETLERİ VE TİC. A.Ş.(Nº: 2163)
CE Certificate nº:	2163 – PPE – 2055



Materials and Components

COMPONENTE	MATERIAL
Outer layer	Polypropylene
2 nd layer	Polypropylene
3 rd layer	Polypropylene
4 th layer	Polypropylene
Inner layer	Polypropylene
Elastic bands	Polyamide and spandex
Nose clipl	Plastic coated metal

It does not contain components made of latex.
It does not contain substances classified as carcinogenic, mutagenic, or toxic for reproduction.

Marking

NR	Not reusable
	Expired date (AAAA/MM)
	Storage temperature range
	Maximum relative humidity of storage
	See instructions for use
	Batch number

INSTRUCTIONS FOR USE

IMPORTANT

It is advisable to check the integrity of the mask before use. If you notice the presence of damage and / or stains, order a new mask.

The NR symbol in the marking indicates that the mask is not reusable and must be discarded at the end of the work shift.

The protection offered by an FFP3 or FFP2 filter mask includes that offered by the FFP1 class.

INSTRUCTIONS FOR PLACING THE MASK:

1. Slightly open the mask and, using a finger, shape the nasal bridge to give it the shape of the nose.
2. Place the mask against your face and hold it by placing the elastic bands over your ears.
3. Using both hands, shape the clip around the nose.
4. Make sure that the mask adheres well to the face - there should be no spaces between the mask and the face.
5. Replace the mask immediately if you have difficulty breathing or if the mask is damaged, or if there are problems with adhesion to the face. Respecting these instructions carefully is essential for the safe use of the respirator.
6. Clean your hands with soap and water or an alcohol-based solution before and after using the mask.

WARNINGS: [PPE - Personal Protective Equipment]

If it is impossible to obtain an adequate adhesion of the mask to the face profile, do not enter environments with potential risk.

Failure to follow the instructions provided can lead to misuse of the PPE and cause illness and death.

This product does not provide oxygen. It should only be used in adequately ventilated environments, with sufficient oxygen content to maintain life.

Do not use when concentrations of contaminants are immediately hazardous to health or life.

Do not use in an atmosphere where there is a danger of explosion.

Beard hair and respiratory conditions can reduce the efficiency of these filter masks. In addition, there may also be a reduction in the performance of the mask if the mask is damp or wet (the mask must be replaced).

Do not modify or alter the filter mask in any way.

For single use only.

Dispose of the mask after each use.

The declaration of conformity can be obtained at www.pharmia.eu.